

Horse DNA Submission Form

Instructions:

1. Please do not write on this form. Contact the registry for corrections.
2. Thoroughly wash and dry your hands.
3. Pull (DO NOT CUT) 20-30 tail or mane hairs by wrapping hairs around forefinger. For foals, use tail hairs only. Grasp hair close to the animal's body to insure roots are included. Pull straight toward your body. Visually check that roots are attached to hair. DO NOT TOUCH ROOT BULBS.
4. Place sample in shaded box (below) with roots at left. Tape in place with tape over hair shaft on right.
5. Fold submission form backwards so that barcode and identification are on the outside and hair is covered. See fold lines for details. Do not seal submission form with tape or staple shut.
6. Place in envelope and mail to the address below.
7. If mailing from outside the United States, a current import permit must accompany the sample.

Mail To:

Veterinary Genetics Laboratory - Horse DNA Testing
One Shields Avenue
University of California
Davis, CA 95616-8744

For FedEx, UPS, DHL, etc:

Veterinary Genetics Laboratory - Horse DNA Testing
Old Davis Road
University of California
Davis, CA 95616-8744

Second fold (Fold back to cover hair. Do not cut, tape or staple)



For VGL use

LHN
DP

FOLD WITH BARCODE ON OUTSIDE

Owner

Name: Les haras nationaux
Address: DCO/SIRE
Route de troche, BP3
19231 Arnac Pompadour cedex
France

Animal being tested

Name: UNFURLED
Registration:
Barn Name:
Color: BRO
Date of Birth: 04/05/2008 Sex: Mare
Breed: French Warmblood (Selle Francais)

Parent Information

SIRE

Name: BALADIN D'OC
Registration: 89550583J
VGL Case #: SFA13

DAM

Name: RUN REALLY RUN
Registration: 9416884
VGL Case #: SFA1032

First fold (Fold back. Do not cut, tape or staple)



Place
Roots
Here

Tape
Here

Place rest of hair here.
Coil if necessary.

Pull 20-30 tail or mane hairs of animal being tested and place above. Only one animal per form.